	<p>Gebr. Brasseler GmbH &amp; Co. KG</p> <p>Trophagener Weg 25</p> <p>32657 Lemgo · Germany</p> <p>Telefon +49 (0) 5261 701-400</p> <p>Telefax +49 (0) 5261 701-580</p> <p>info@kometmedical.de</p> <p>www.kometmedical.de</p>
	

**Rotierende Instrumente steril**  
© Gebr. Brasseler GmbH & Co. KG  
**302853 - Rev 2019-03**

**(DE) (EN) (FR) (ES) (IT) (ZH)**

	<p>Bei beschädigter Verpackung nicht verwenden,  Do not use in case of damaged packaging, Ne pas utiliser si l'emballage est endommagé, No utiliziar si el embalaje está dañado, Non utilizzare se l'imballo è danneggiato, <b>如果包装破损请勿使用</b></p>
<b>(DE)</b>	

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Instructions for use and safety recommen-dations for the application of sterile rotary instruments in the medical field

The instruments are made of high-grade materials. They are designed, constructed and produced with the utmost care. These quality instruments assure best working results and long service life provided they are used in the proper manner. Therefore, the following instructions for use and safety recommendations have to be observed. Improper use can lead to damage to the tissue, premature wear, destruction of the instruments and injury to the operator, patient or other persons.

- Proper use
  - Observe this information and the instructions for use for surgical motor systems when using rotary instruments.
  - Before use, make absolutely sure that the instruments to be used and their packaging are in sterile and in technically sound condition. Discard any damaged instruments.
  - Instruments with clamping marks (line or ring) have to be lowered into the hand-piece until the marks are no longer visible.
  - Insert rotary instruments in the hand piece until the stop position is reached.
  - Prior to operation, make sure that the instrument is firmly seated.
  - Do not use the instrumentes for working on metals (e.g. implants) and avoid any contact with metal during application.
  - Craniotomes are to be used with corresponding dura guard only.
  - To ensure traceability of the MD by means of the UDI, please make sure to keep the packaging of articles without direct marking.

## 2. Indication

The instruments have been designed for working on bone, skin, soft tissue and cements. Instruments of adapted materials are available for each application.

- Dermabrasion: instruments with diamond-coated working part
  - Maxillo-facial surgery: instruments made of stainless steel, tungsten carbide instruments, diamond instruments
  - ENT surgery: instruments made of stainless steel, tungsten carbide instruments, diamond instruments
  - Neurosurgery: stainless steel, carbide and diamond instruments
- The surgeon is responsible for the selection of the appropriate instrument.

## 3. Contraindication

The instruments are not suited for use in the CNS (brain and spinal cord).

## 4. Contact pressure

- Excessive contact pressure has to be avoided.
- It can lead to damage of the instruments' working part (steel and tungsten carbide instruments) as well as damage to the blades. At the same time extreme heat is generated which increases the risk of thermal necrosis.
- Excessive contact pressure can lead to stripping of the abrasive grit of diamond instruments. Increased heat development will be the result.
- At the worst, breakage of the instrument can occur if too high contact pressure is applied. Jamming or leverage of the instrument must be avoided during operation. Optimal material reduction is assured at a contact pressure of approx. 2N. This can be compared with the pressure which is applied when writing with a fountain pen.

## 5. Cooling

When using rotary instruments sufficient cooling has to be supplied, otherwise the instrument blades will be clogged with chips which lead to higher heat generation and at worst to irreversible damage to the bone (thermal necrosis). Moreover, service life of the instruments will be reduced.

<b>(FR)</b>	
-------------	--

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## Instructions for use

Please read carefully and retain for future reference.

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- Disposal of blunt and damaged instruments
  - Check instruments for blunt and damaged blades using magnifying glasses. Check especially for:
    - Chipped and blunt blades
    - Blank spots on diamond instruments
    - Damage to the shank
    - Bent or "out of true" instruments
  - Blunt and damaged instruments must not be reused under any circumstances. Otherwise instrument breakage and risk of injury to the patient and operator may be the result.

## 7. Cleaning, disinfection, sterilization and storage

- A validated reprocessing method is available on our web site www.kometmedical.de under "Instrument reprocessing".
  - This instrument comes in sterile packaging and does not require processing prior to first use, provided that the sterile packaging is intact. Add patient labels to the patient/hospital record etc. when the sterile packaging is opened for the first time. The manufacturer regrets being unable to guarantee sterility once the instruments have undergone a reprocessing cycle.
  - The instruments should be placed in hygienic bur blocks, dishes or other suitable containers. The same applies to sterilized instruments and instruments in sterile packages. The instruments are to be stored in an environment free of dust and humidity guaranteeing no recontamination.
  - After surgical operation, immediately remove blood, secretion, tissue and bone residues. Dried residues can cause corrosion.
  - Make sure that all residues are removed; this applies especially to hollow burs.
  - The instruments can be cleaned in the ultrasonic bath. However, care should be taken that the instruments do not come in contact with each other; otherwise the blades can be damaged.
  - It is recommended to use anticorrosive disinfection and cleaning agents (e.g. Komet DC1).
  - Contact with H<sub>2</sub>O<sub>2</sub> (hydrogen peroxide), chlorous disinfection and cleaning agents as well as oxalic acids are to be avoided because these agents may lead to pitting and corrosion.
  - For immersion time and suitability of disinfection and cleaning substances for specific instrument types see instructions for use of the manufacturer of these agents.
  - Disinfection and cleaning agents must be rinsed off thoroughly under running water immediately after use; otherwise corrosion can occur due to chemical reactions.
  - Disinfection of rotary instruments can be carried out in the gentle cycle (e.g. Vario TD) of the washer/ disinfecter. Use cleaning and disinfection agents with approved material compatibility. We recommend deconex 28 AlkaOne (co. Borer Chemie).
  - Sterilization should take place in the steam sterilizer.
  - A temperature above 180°C has to be avoided. Exceeding this temperature would result in loss of hardness of the working part and thus reduction of service life.
- Make sure that each selected disinfection and sterilization method is suitable for the specific instruments. The instruments may only be returned in clean, disinfected and sterilized condition.

## 8. Safety and liability

- For disposable instruments: Due to the wear of the material, a safe and risk-free reuse of the product cannot be guaranteed.
- Please observe the instructions in the up-to-date Komet Medical brochure on instrument reprocessing.
- The user is responsible for checking the product prior to use to ensure that it is suitable for the intended purpose.
- The user is responsible for the use of the instruments.
- In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

<b>(FR)</b>	
-------------	--

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