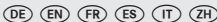
Komet Medical
Gebr. Brasseler GmbH & Co. KG
Trophagener Weg 25
32657 Lemgo · Germany
Telefon +49 (0) 5261 701-400
Telefax +49 (0) 5261 701-580
info@kometmedical.de
www.kometmedical.de

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Instructions of use/cleaning instructions for stainless steel bur blocks

1. Range of application

The Komet Medical bur blocks allow the compilation of surgical instrument sets according to individual requirements. In order to facilitate the preparations for the treatment, the bur block can be fitted as desired with frequently used or other required instruments.

9853M: 24 burs, cutters and abrasives with a shank diameter of 2.35 mm and a total length of 70 mm.

9985M: 14 burs, cutters and abrasives with a shank diameter of 2.35 – 3.16 mm and a total maximum length of up to 80 mm.

The instruments are easily recognised and removed from the block, thanks to the conveniently distanced boreholes as well as the inclined position of the instruments.

2. Material

The Komet Medical bur block is composed of several components that are screwed together tightly. The metals used for the individual components meet all the expectations placed in high quality, stainless instrument steel.

3. Cleaning

Brand new bur blocks must be processed prior to first use. In order to prevent residues from drying on the instruments, the Komet Medical bur block should be clean-ed with appropriate cleaning agents immediately after use. Remove stubborn contamination with a plastic brush which must not be too hard. To avoid damage to the material, rinse the Komet Medical bur block thoroughly under running water and dry with compressed air. The mechanical reprocessing of the Komet Medical bur block takes place in the washer/ disinfectant. After completion of the cleaning process check the Komet Medical bur block to ensure perfect condition and function. Make sure that the bur block is in open condition when it is placed in the washer/disinfectant and that all instruments are reached and washed by the spray jet.

9985M: The removable sieve must be placed in the washer/disinfectant separately, to ensure that the support surfaces of the sieve are cleaned and disinfected.

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4. Sterilization

The Komet Medical bur block has to be cleaned prior to sterilization (as described above). The Komet Medical bur block can then be sterilized, either with or without instruments. The bur block is to be placed into a container suitable for the option chosen. The Komet Medical bur block may only be sterilized according to an approved and validated method of sterilization.

5. Precautions/warnings

- Only use suitable cleaning and disinfecting agents approved by the VAH for manual cleaning.
- The instructions provided by the manufacturer with regard to material compatibility, concentration and immersion time are to be observed.
- Check the Komet Medical bur blocks prior to each use to ensure perfect condition and function.
- Remove any stains immediately after cleaning and disinfecting with a cloth.
- Once taken out of the cleaning or disinfecting agent, rinse Komet Medical bur block thoroughly under running water and dry (for example by air jet), paying particular attention to the bore holes.
- Do not sterilize the Komet Medical bur block until it is clean and completely free of any remnant cleaning and disinfecting agent.
- The sterilization of the Komet Medical bur block must take place in a type B sterilization by vapour at a temperature of 134°C.

6. Safety and liability

- Please observe the instructions in the up-to-date Komet Medical brochure on instrument reprocessing.
- The user is responsible for checking the product prior to use as to whether it is suitable for the intended purpose.
- The user is responsible for the application of the instruments.
- In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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